

- QODOA-156NTEUO-I** 5 Panel T-Square no THC with Indicator; AMP/COC/MET/OPI/PCP \***NO THC**
- QODOA-666EUO-I** 6 Panel T-Square with indicator, AMP/COC/MET/OPI/OXY/THC
- QODOA-376EUO-I** 7 Panel T-Square with indicator; AMP/COC/MET/OPI/OXY/PCP/THC
- QODOA-10106EUO-I** 10 Panel T-Square with indicator, AMP/BAR/COC/MDMA/MET/MTD/OPI/OXY/PCP/THC ★

- **Compact, built-in saturation indicator with swab collection device**
- **Configurable. Up to 12 customizable drugs on one device**
- **Greatly decreases ability to adulterate sample**
- **Excellent in a "reasonable suspicion" scenario**
- **Available for forensic use, employment & insurance**


★ - Best Seller




# T-Square

## Multi-Drug Oral Fluid Test

19525 Janacek Ct, Suite 104 Brookfield, WI 53045  +1 262.432.9308

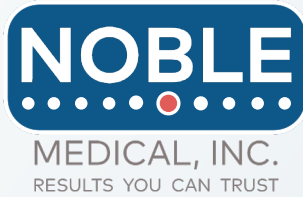

- **Compact, built-in saturation indicator with swab collection device**
- **Configurable. Up to 12 customizable drugs on one device**
- **Greatly decreases ability to adulterate sample**
- **Excellent in a "reasonable suspicion" scenario**
- **Available for forensic use, employment & insurance**

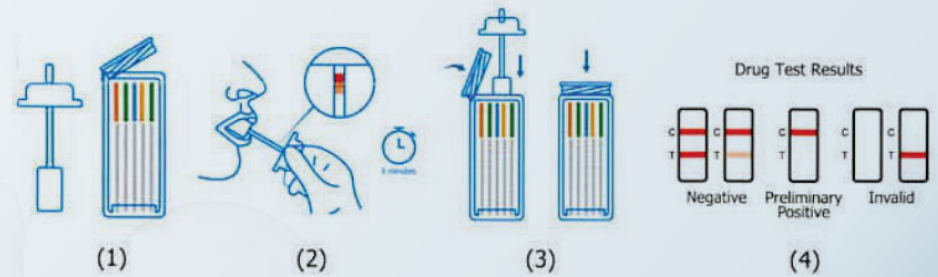

#### Directions for use

Allow the kit and specimen to come to room temperature (65°F-86°F/8°C-30°C) prior to testing. **AVOID PLACING ANYTHING IN THE MOUTH 10 MINUTES PRIOR TO TESTING.**

1. Remove the test cube/square and the sponge collector from the foil pouch by tearing at the notch. Place the test cube/square upright on a level surface.
2. Put the sponge end of the collector on your tongue or near cheek to collect oral fluid for about 3 minutes until color on saturation indicator strip appears RED in the indicator window. If color on saturation indicator has not turned red at 7 minutes, repeat the collection using a new device, beginning with Step 1.
3. Open the test cube/square and place the saturated oral fluid collector inside the test cube/square. Press the sponge collector down firmly until it reaches the bottom of the test cube/square then tightly close the cube/square lid. Keep test cube/square upright on flat surface and follow Step 4.
4. Interpreting Drug Test Results: Read results at 5 minutes. Do not read after 5 minutes.



## T-Square Multi-Drug Oral Fluid Test

19525 Janacek Ct, Suite 104 Brookfield, WI 53045  +1 262.432.9308



The Noble Group