

NMCup-04-18	4 Panel Cup AMP500/COC150/MET500/OPI300 *NO THC
NMCupD-05-02-BT	5 Panel Cup w/ Adulteration Strip, AMP500/COC150/MET500/OPI300/THC50 (CRE, pH,OX)
E-NMCup-09-20-BT	9 Panel Cup, 6AM10/AMP500/COC150/MDMA500/MET500/OPI300/OXY100/PCP25/THC50
NMCupD-10-16-BT	10 Panel Cup w/ Adulteration Strip AMP/500/BAR300/BZO300/COC150/MET500/MTD300/OPI300/OXY100/PCP25/THC50 (CRE, pH, OX)
E-NMCup-11-25-BT	11 Panel Cup AMP500/BAR300/BZO300/COC150/MET500/MTD300/OPI300/OXY100/PCP25/PPX300/THC50
NMCup-11-37	11 Panel Cup AMP500/BAR300/BUP10/BZO300/COC150/MDMA500/MET500/MTD300/OPI300/OXY100/PCP25 *CLIA WAIVED
NMCup-12-75-BT	12 Panel Cup AMP500/BAR300/BUP10/BZO300/COC150/MAMP500/MDMA500/MTD300/OPI300/OXY100/PCP25/TCA1000 *CLIA WAIVED
NMCup C-12-44-BT	12 Panel Cup w/Adulteration Strip, AMP500/BAR300/BZO300/COC150/MDMA500/MET500/MTD300/OPI300/OXY100/PCP25/PPX300/THC50 (OX/pH/SG) (GL/NI/CR)
NMCup-12-01-BT	12 Panel Cup AMP500/BAR300/BUP10/BZO300/COC150/MET500/MDMA500/MTD300/OPI300/OXY100/TCA1000/THC50 *CLIA WAIVED
NMCup-12-02-BT	12 Panel Cup AMP500/BAR300/BUP10/BZO300/COC150/MDMA500/MET500/MTD300/OPI300/OXY100/PCP25/THC50 *CLIA WAIVED
NMCupD-12-06-BT	12 Panel Cup w/Adulteration Strip AMP500/BAR300/BZO300/COC150/MDMA500/MET500/MTD300/OPI300/OXY100/PCP25/TCA1000/THC50 (Cre, pH, OX)
NMCup-13-01-BT	13 Panel Cup AMP500/BAR300/BUP10/BZO300/COC150/THC50/MET500/MDMA500/MOP300/MTD300/PCP25/TCA1000/OXY100 *CLIA WAIVED


- **Accurate**
- **Affordable**
- **Min. Vol. To Run is 5 mL**
- **Negative Results in > 1 minute**
- **Available in CLIA Waived 501K, Forensic Use, Insurance**



Noble Drug Test Cup

For Rapid Drug & Alcohol Screening

19525 Janacek Ct, Suite 104 Brookfield, WI 53045  +1 262.432.9308

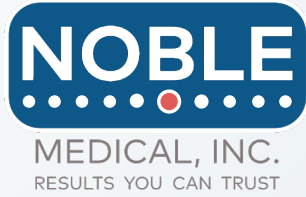


- **Accurate**
- **Affordable**
- **Min. Vol. To Run is 5 mL**
- **Negative Results in > 1 minute**
- **Available in CLIA Waived 501K, Forensic Use, Insurance**

Directions for use


Step 1
Remove the cup from the foil pouch.




Step 2
Collect the urine in the cup.

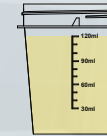


The maximum amount of urine (120mL).

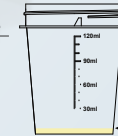



The minimum amount of urine required (5mL).

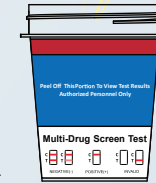

Step 3
Secure cap tightly.

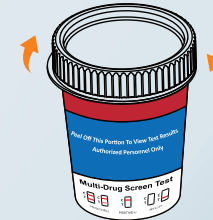

Step 4
At 5 minutes, peel the label and read the results. Do not read after 60 minutes.

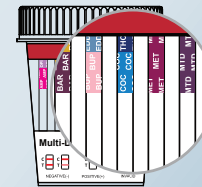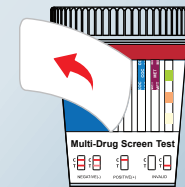

Noble Drug Test Cup

For Rapid Drug & Alcohol Screening

19525 Janacek Ct, Suite 104 Brookfield, WI 53045 +1 262.432.9308

